


Instructions for using sunglasses

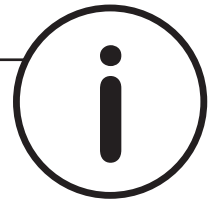


Information from the manufacturer

These sunglasses comply with Regulation (EU) 2017/745. For Europe, the standard is ISO 12312.1, and in the USA, according to the national standard, ANSI Z80.3. The declaration of conformity is available at <https://johann-v-goisern.com/documents/>. The lenses used correspond to filter category 3. I.e., the lenses are dark tinted, suitable for light transmittance 8-18% and have 100% UV protection.




Areas of use

Your sunglasses protect against glare from sunlight, against natural UV radiation (100% UV protection), in snow, rain and wind. Not suitable as protection against impact, hard objects, artificial light sources such as lamps with high radiant energy (UV lamps) and solariums (sun lamps). For artificial light sources you need special protective glasses. In addition, do not look directly at the sun with these sunglasses and do not use them to observe a solar eclipse, either with or without sunglasses. Do not wear sunglasses for driving at night or in poor visibility conditions. Remove your sunglasses when the light level is too low to see well.

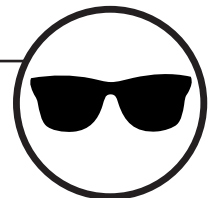

Storage and transport

The sunglasses should be stored in their case at a temperature between -10°C and +35°C. Storage under sunlight in closed spaces without air conditioning (e.g., car dashboard) should be avoided, as the temperatures reached under these conditions may affect the functionality of the frame (deformation of the frame).

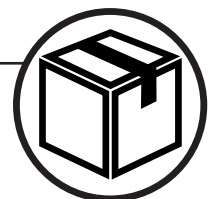

Instructions for using sunglasses



Cleaning

For optimal function, clean your glasses regularly with clean water and a soft cloth. A mild soap, dishwashing detergent or commercially available eyeglass cleaning agents can also be used. In case of contact with reagents such as salt water (seawater), swimming pool water and chemical products (e.g., cosmetics, hair spray, sun creams and insect repellents), immediate cleaning with clear water and subsequent drying with a soft cloth is recommended. In case of wear (e.g., scratches, loss of gloss), replacement with original spare parts is recommended. Unclean cleaning cloths may impair the filter properties.

The use of solvents (e.g., alcohol and acetone) and aggressive chemical cleaning agents, which could affect the functionality of the glasses, should be absolutely avoided.



Product warranty

National warranty and guarantee regulations apply.



Labelling

On the packaging of your product, in addition to the model and colour number, there is also a UDI-PI (Unique Device Identification - Production Identifier) and a UDI-DI (Unique Device Identification - Device Identifier) number. These numbers are used to identify, mark, and register medical devices. The goal is to increase patient safety through complete traceability of products.



Manufacturer

Pomberger Goisern GmbH
Au 131, 4822 Bad Goisern a. H., Austria
UID: ATU78604627
Company registers number: 590054 b
ARA license number: 4826
Website: www.pomberger.com

Contact Information

If your optician where the product was purchased cannot be contacted, you can contact us with complaints or questions by email at office@pomberger.com or by phone at +43 (0) 6135 7465.